STOMA OIL

INSTRUCTIONS FOR USE:

Stoma oil is a colourless neutral oil used for the lubrication of tracheal cannulas.

Please apply the stoma oil sparingly and spread a few drops carefully over the outer cannula. Under no circumstances use a dripping wet cannula!

In order to prevent bacterial contamination, hygienic precaution is necessary when using the product. Before applying the stoma oil, please wash your hands thoroughly (or use surgical gloves). Furthermore, only use a clean and non-fuzzing wipe for applying the oil to the cannula. Do not touch the bottle opening with your fingers!

Stoma oil should only be used by one single patient.

Attention!

Do not apply the stoma oil directly to the tracheostoma!

Lubricating the cannula with stoma oil facilitates its insertion and makes it more comfortable to wear. The dosage bottle allows for a well-dosed, targeted application of the stoma oil.

Please make sure not to use the oil after the "use by" date (see bottom of bottle) has expired. After the first opening of the bottle, the stoma oil can be used for up to 30 days.

Stoma oil can be used for silver and for plastic cannulas. Before using the stoma oil, please make sure to observe the cannula manufacturer's instructions for care and maintenance of your individual cannula.

Stoma oil is also available in single dosages of 4 ml, in form of an Optifluid® stoma oil wipe.

The use of this product variant is recommended when travelling and for clinics and public institutions.

Ingredients:

Various triglycerides (neutral fats)

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