# Cuticell® Classic

## Paraffingaze

## Paraffin gauze dressing

## Compresse de gaze paraffinée

## Vendajes de gasa parafinada

## Garza grassa alla paraffina

## Paraffine vetgaaskompres

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### Hersteller:

BSN medical GmbH & Co. KG, D-22771 Hamburg

(GB) Cuticell® Classic is a sterile, non-medicated, paraffin gauze dressing with open weave.

### Properties:

Due to its paraffin coating, Cuticell® Classic is soothing and low-adherent. The open weave allows exudate to pass through into an absorbent secondary dressing.

The gauze has interlocking threads which minimise fraying when the dressing is cut to shape.

### Indications:

Cuticell® Classic is used as a primary wound contact layer in the treatment of exuding wounds such as

- Minor burns and scalds
- Donor and recipient skin graft sites
- Skin loss wounds
- Lacerations and abrasions
- Leg ulcers

### Instructions for use:

Prior to application, the wound should be cleaned as usual. Remove Cuticell® Classic from the peel pack and cut it to the size of the wound if necessary.

To absorb wound exudate, a sterile absorbent pad can be placed over Cuticell® Classic. For fixation, bandages or adhesive material can be used. Cuticell® Classic should be removed at the latest as soon as the paraffin starts to liquefy.

### Assortment:

Cuticell® Classic is available in different dimensions and pack sizes:

Size	Dressings/pack	REF No
5 x 5 cm	5	72538-00
5 x 5 cm	50	72538-01
10 x 10 cm	10	72538-02
10 x 10 cm	100	72538-03
10 x 40 cm	10	72538-04
10 x 10 cm (dispenser)	36 (not individually wrapped)	72538-05
7 m x 10 cm (dispenser)	1	72538-06

### General instructions/Cautions:

Contents sterile unless peel pack is damaged. For single use only.

Do not re-sterilise.

### Manufacturer:

BSN medical GmbH & Co. KG, D-22771 Harburg, Germany

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BSN medical GmbH & Co. KG, D-22771 Hamburg, Germany